



DollyPHC Installation Instruction Sheet

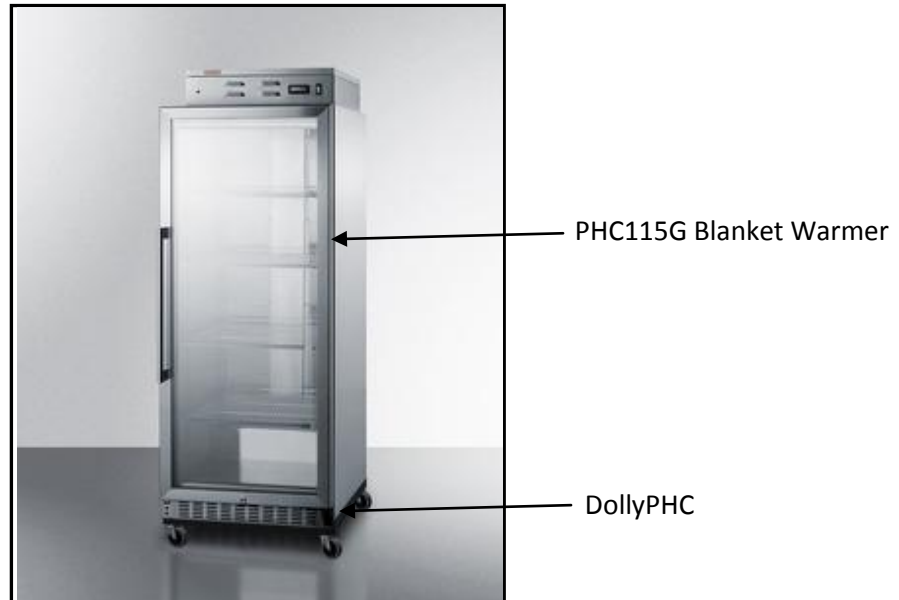

To make the PHC115G blanket warmer easy to move, you will need to attach DollyPHC by following the instructions below. *NOTE: This requires the assistance of more than one person.*

1. On the rear of the PHC115G, you will see a plate towards the bottom of the unit with four screws holding it in place. Remove the two lower screws and their washers. You will use these pieces later.
2. Set the dolly down and lock the casters. (To lock, just push on the lever at the side of each wheel. To unlock, push on the lever again.)
3. Carefully lift and place the PHC115G on the dolly. The back of the unit should align with the back of the dolly where you will see two holes.
4. Screw the two screws and washers removed in Step 1 through the two holes in the dolly and into the screw holes on the bottom of the blanket warmer. This will secure the dolly to the PHC115G.
5. Unlock the casters and roll the blanket warmer into its desired location.

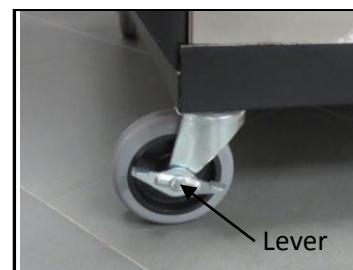

Once the unit has been rolled into place, you can lock the casters again to keep it from moving. However, if the PHC115G will be located in a place that is hard to reach, **DO NOT LOCK** the two rear casters because this would make it difficult to move the unit at a later time.